

# PTS Carrier Packing Guide

- Specimens must first be properly packaged before transit in the tube system.
- Improperly packaged samples are damaged during transit and may impact patient results, or may be unacceptable for testing.



- ✓ Ensure both carrier latches are fully closed and secured.
- ✓ Both foam liners must be used to fully immobilize contents.
- ✓ Ensure contents are entirely contained and nothing sticks out.

- ✗ If contents do not fit inside the tube, please walk package to the destination.
- ✗ If carrier is missing foam, please send the empty tube to lab for replacement.
- ✗ If carrier is broken or damaged, report to facilities at x88351 and set aside.

